

VERMEIREN

378

USER MANUAL



Content

Content	1
1 Preface	1
2 Your product.....	1
3 Before use.....	1
4 Assembly and adjustments	2
5 Using your table	3
6 Maintenance.....	3
7 Technical details.....	4

1 Preface

Thank you for your trust in the products of Vermeiren. To support you on the use of this product and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your product.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

Pictures of the product are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website, <http://www.vermeiren.com/>, you will always find the most recent version of the information in this manual. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

2 Your product



1. Table tray
2. Ledge
3. Lower frame
4. Castors with brakes
5. Wing nut for height adjustment
6. Star knob for depth adjustment
7. Identification plate

3 Before use

3.1 Upon delivery

Unpack your product and verify if the delivery is complete. The following items should be included:

- Lower frame (including 4 castors with brake);
- Upper frame (incl. angle adjustable table tray);
- Manual.

Verify your product for transport damage. If you find any damages after delivery, contact the transporter.

3.2 Intended use

- This table is used as a tray for holding various objects when sitting in your comfort chair, shell chair or lift chair.
- The maximum load of the table is 10 kg.
- The table is meant for indoor use on level surfaces. When in use, check that all 4 castors touch the floor.
- It is not intended to support or transport goods or persons.
- The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of

maintenance will cause the warranty to lapse.

3.3 Safety instructions



CAUTION

Risk of injury and/or damage

- Do not stand on the tubes of the lower frame, the table could tip over.
- Do not lean against or on the tray.
- Do not lift the table by the tray.
- Do not place any objects larger than the table on the tray.
- Do not exceed the bearing capacity of the table.
- Do not transport persons or objects using the table.
- Repair work may only be undertaken by an authorised dealer.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

3.4 Shipping and storage

The shipping and storage of the table should happen according to the technical specifications in §7. Make sure that the table is stored dry.

Provide sufficient covering or packaging to protect the table from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

Make sure no objects are stored on or against the table to prevent damage.

4 Assembly and adjustments

4.1 Assembly

The table is delivered fully assembled. The wing nuts and star knobs must be tightened before any objects can be placed on the trays.

4.2 Adjustments



CAUTION

Risk of injury

- **The stability of the product is affected when the tray is adjusted (danger of tipping over).**
- **The tray should only be tilted so far that nothing will fall off. For your safety, a ledge has been fitted.**

- **When inclining the table, make sure that there is enough space between the table and the chair and/or other objects.**



4.2.1 Height adjustment

The height adjustment is indicated with mark lines (5) on the tube.

1. Loosen the wing nut (1) on both sides.
2. Move the table tray (4) upwards or downwards until the desired height (range 180 mm: steps 10 mm).
3. Retighten the wing nuts (1). Check that the height adjustment for both sides is firmly secured to the same position.

4.2.2 Depth adjustment

1. Loosen the star knob (2).
2. Move the lower frame to the front / back until the desired depth (range 250 mm: stepless).
3. Retighten the star knob (2) and repeat for the other side. Check the depth adjustment is firmly secured at both sides.

Minimum depth of insertion: 50 mm.

4.2.3 Angle adjustment


1. Pull the strap (3) below the table tray (4).
2. Keep pulling the strap (3) and turn the tray (4) upwards or downwards until the desired angle (range 90°: 6 positions, steps of 18°).
3. Release the strap (3). Check that the angle adjustment is firmly secured.

5 Using your table

5.1 Positioning around the chair




5.2 Operating the brakes

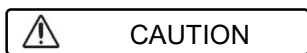

Risk of injuries and/or damage

The brakes can wear. Check the working of the brakes before each use.

To put the table in parking position, all four wheels must be locked.

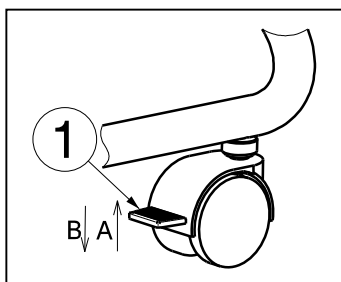

5.2.1 Engaging the brakes [B]

1. With your foot, press the brake plate downwards until it locks in place.
2. Repeat for the other wheels. The wheels can now neither roll nor turn.

5.2.2 Disengaging the brakes [A]

1. With your foot, press the brake plate upwards.
2. Repeat for the other wheels. The wheels have now been released and can both roll and swivel.

6 Maintenance

6.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap to clean the table. Avoid soaking. Do not use abrasive cleaning agents to clean.

6.2 Disinfection



Risk of damage

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

Use a soft, damp cloth and lukewarm water with a mild household disinfectant to clean the table.

6.3 Inspection / Service / Repairs

The expected service life of your table is 5 years. The lifespan is influenced by use, storage, maintenance, service and cleaning.

Before you resume use of the table, it should be serviced according to the following criteria:

- Completeness
- Adjustment functions
- Castors (stability, tracking, fixing screws)
- Fastenings and adjusting buttons: well secured (Condition, thread, holding security)
- Frame (deformation, stability)
- Tray (closed surfaces, splinters, tears)
- Damaged varnish (remove possible rust and repaint)

Repairs and replacements may only be undertaken by trained persons, and only genuine replacement parts of Vermeiren



should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

6.4 Reuse

Before each reuse, have the table disinfected, inspected and serviced according to the instructions in §6.

6.5 Disposal at end of life

At end of life, you need to dispose of your table according to the local environmental legislation. The best way to do so, is to disassemble the table to facilitate the transport of recyclable parts.

7 Technical details

Make	Vermeiren
Type	Table
Model	378

<i>Description</i>	<i>Dimensions</i>
Max. load	10 kg
Total height	640 - 820 mm
Height table	620 - 800 mm
Length	950 mm
Width	600 - 850 mm
Angle table tray	0° - 18° - 36° - 54° - 72° - 90°
Height lower frame	215 mm
Free height above the floor	60 mm
Table area (L x W)	950 x 500 mm
Weight	11,30 kg
Diameter wheels	50 mm
Storage temperature	+5°C - +45°C
Storage humidity	20% - 75%
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	



Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:



Vermeiren GROUP
Vermeirenplein 1 / 15
2920 Kalmthout
BE

website: www.vermeiren.com

Instructions for specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, 2020-07

Basic UDI: 5415174 1803378 LS

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

© Vermeiren Group 2020