VERMEIREN

253

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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EN Contents

1. Technical details	2
2. Components	2
3. Explanation of symbols	2
4. Check after delivery	2
5. Intended use	2
6. Fold and unfold the rollator (Fig. 1)	3
7. Assembly	3
8. Pushing handgrips (Fig. 2)	3
9. Basket	3
10. Safety instructions	4
11. Care	4
12. Disinfection	4
13. Repairs / Service / Inspection	4
14. Shipping and storage	5
15. Warranty	5
16. Disposal	5
17. Declaration of conformity	5

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Rollator 253

1. Technical details

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

Make	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Rollator	
Model	253	
Width	560 mm	
Length (unfolded)	670 mm	
Length (folded)	185 mm	
Maximum height	915 mm	
Minimum height	795 mm	
Seat height	530 mm	
Seat width	415 mm	
Seat depth	250 mm	
Weight	4,60 kg	
Weight tray	0,25 kg	
Weight basket	0,85 kg	
Wheels	Ø 145 mm x 25 mm	
Max. inclination forwards	15°	
Max. inclination backwards	7°	
Max. inclination sideways	3,5°	
Maximum user weight	max. 130 kg	
Maximum basket weight	max. 5 kg	
Turning width	780 mm	
Storage temperature	+5°C - +41°C	
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / °		

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.


If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren

facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can contact the dealer for the instructions for use.

2. Components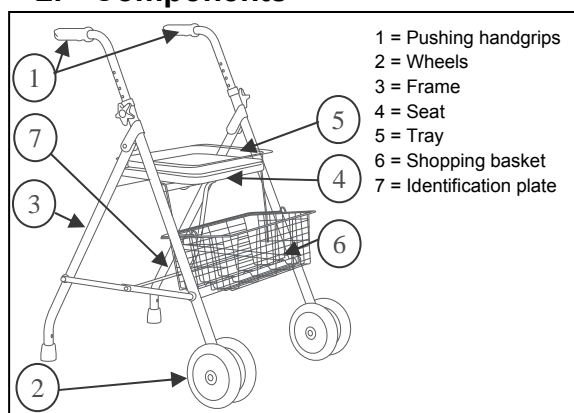

3. Explanation of symbols

Maximum mass

CE conformity

Type designation

4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- rollator frame with seat, 4 wheels (mounted) and two tips.
- height adjustable pushing handgrips
- shopping basket, tray
- manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

5. Intended use

⚠ WARNING: Risk of falling - Do not use the rollator if you have weak arm function or poor equilibrium.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide a better stability, support and safety during walking.

However, it cannot be used by persons with a visible weakened hand and/or arm function and also not by persons with a balance problem.

The rollator is only for **indoor** use.

The rollator should only be used on smooth surfaces. Avoid wet floors. It is recommended to remove the carpets out rooms where using your rollator. The product is not meant for transport of people and/or goods. The seat can only be used when the rollator stand on stable surfaces.

6. Fold and unfold the rollator (Fig. 1)

⚠ WARNING: Risk of clamping - Keep fingers, buckles and clothes away from the folding mechanism.

1. Folding rollator


- Remove the tray.
- Lift the seat upwards.
- Remove the shopping basket.
- Pull the black cord ① upwards.
- Push the rear tubes ② forwards against the front tubes to fold the rollator further.

2. Unfolding rollator

- Pull the rear tubes ② backwards to unfold the rollator.
- Push the attachment plates ③ on both sides downwards until these lock in position.
- Lift the seat upwards, if necessary.
- Place the shopping basket on the horizontal tubes were the seat rest on.
- Turn the seat downwards.
- Place the tray.

• **Adjust the pushing handgrips to the desired height (see further).**

Fig. 1 – Folding / Unfolding rollator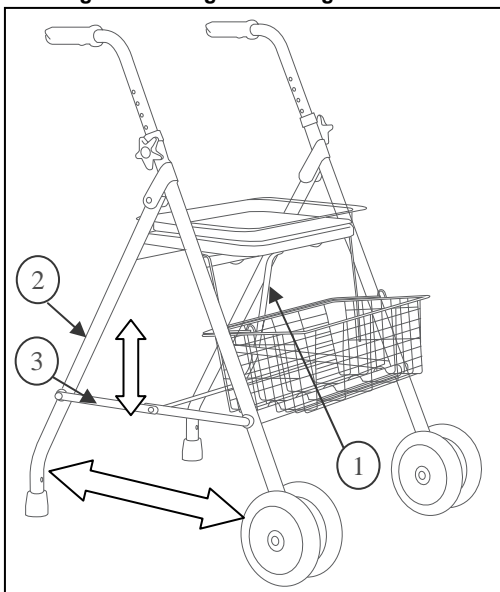

7. Assembly


- Slide the pushing handgrips in the front frame.
- Unfold the rollator.
- **Adjust the pushing handgrips to the desired height (see further).**

8. Pushing handgrips (Fig. 2)

⚠ CAUTION: Risk of falling - Secure the handgrips firmly before using the rollator.

⚠ CAUTION: Risk of falling - Do not exceed the maximum height adjustment of the handgrips, your rollator will not be stable anymore.

Fig. 2 – Adjustment pushing handgrips

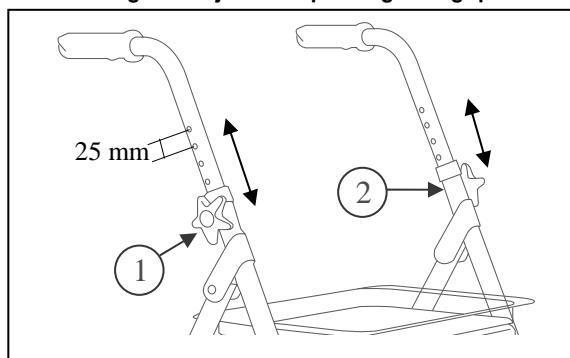The pushing handgrips can be adjusted in height by following steps:

- Loosen the star button ①.
- Remove the star button ① completely.
- Remove the bolt ② out the tube.
- Adjust the pushing handgrips to the desired height (7 different positions: steps from 25 mm). When standing straight, the handgrips should be as high as your wrists. Leave your arms hanging loose.
- Replace the bolt ② in the hole of the tube.
- Place the star button ① on the bolt ②.
- Retighten the star button ① firmly by hand.
- Adjust the other pushing handgrip with the same instructions.
- Check that the two pushing handgrips are adjusted with the same height.

9. Basket

⚠ WARNING: Risk of falling, tipping over - Do not overload the shopping basket.

The shopping basket is meant to be mounted on the front side. Mount the shopping basket with the two hooks to the horizontal front tube were the seat rest on.

The maximum carrying weight from the shopping basket is 5 kg.

10. Safety instructions

- Ensure there are no objects blocking the regular usage of the wheels of the rollator.
- Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- Before every usage, check that the star buttons for your handgrips are firmly locked.
- Verify if the safety against unwanted folding/unfolding is functioning properly.
- Only proceed moving with your rollator when you are standing comfortably and when both your hands are resting on the pushing handgrips.
- Do not hold the pushing handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- When you detect that your body is getting tired, stop your rollator by placing the tips on the ground and have a seat on the bench.
- **The maximum users weight for your rollator is 130 kg** and can only be used on a flat underground.
Do not use your rollator on a staircase or bumps higher than 40 mm.
- Avoid roads with a very steep inclination.
- Avoid wet floors.
- Do not use your rollator to transport people or items.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

11. Care

It is recommended to use a soapy solution for cleaning. The cleaning liquid should in any case have a pH value of at least 6. Do not use a cleaning agent which contains solvents. Observe the instructions of the cleaning liquid used. Use a damp cloth and lukewarm water for cleaning. Avoid soaking. Do not use metal brushes or any other sharp-edged or pointed cleaning utensils. In the case of damage to the varnish or the woodwork, consult your specialist dealer. Regular care would ensure that your rollator is preserved in a perfectly functional condition.

12. Disinfection

For sterilisation use only disinfectants suitable for treating varnished metal and wood surfaces. Observe the instructions of the cleaning liquid used. Use only proved processes (for scrubbing disinfection) and disinfectants included in the list of the Robert Koch Institute (information can be obtained from: www.rki.de). Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

13. Repairs / Service / Inspection

Lifetime of the rollator is influenced by its use, storage, regular maintenance, servicing and cleaning.

You should use only genuine VERMEIREN replacement parts. Repairs and renovations may only be undertaken by trained persons. Make use of the services offered by your specialist dealer. He would gladly assist you further in regard to service and repairs.

Before you resume use of the rollator, it should be serviced according to the following criteria:

- Completeness
- Folding mechanism
- Check the castors (stability, tracking, fixing screws)
- Frames (deformation, stability, connections)
- Pushing handgrips (safe locking from height adjustment system, good support from the handgrip)
- Seat, shopping basket on wear or damage
- Tips on wear or damage and replace if necessary. To avoid damages on floors.
- Cleaning
- **Damaged varnish (remove possible rust and repaint)**

Every 8 weeks:

- Lubricating (moving parts shall be greased)

Every 6 months or for every new user, it should be serviced according to the following criteria:

- General review
- Disinfection

14. Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- The rollator must be stored without being subjected to strains. (Do not put too heavy parts on the rollator, not clamping between something, ...)

15. Warranty

Excerpt from the "General Business Conditions".

(...)

5. The terms of the guarantee may differ from country to country. Consult your specialist dealer for the guarantee period for warranty claims.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

16. Disposal

When disposing of the rollator, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

17. Declaration of conformity

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V

Address :

Vermeirenplein 1/15
2920 Kalmthout
Belgium

declares under his sole responsibility that the CE marked devices :

Productgroup:	Rollators
Productgroup (GMDN):	Walker, foldable (GMDN 37951)
FAMHP (FAGG) registration number:	BE/CA01/11/2-02965-26-CLI
Brand:	Vermeiren
Type:	251, 253, 275, 275BIS, 266, 286B, 286D, 286E, Streeter, 201 // Delta, 4-light, ECO-Light, Quadri-Light

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC: 2007

and is in conformity with the relevant European harmonized standards:

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ENGLISH

CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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All rights reserved, including translation.
Vermeiren Suisse S.A.