



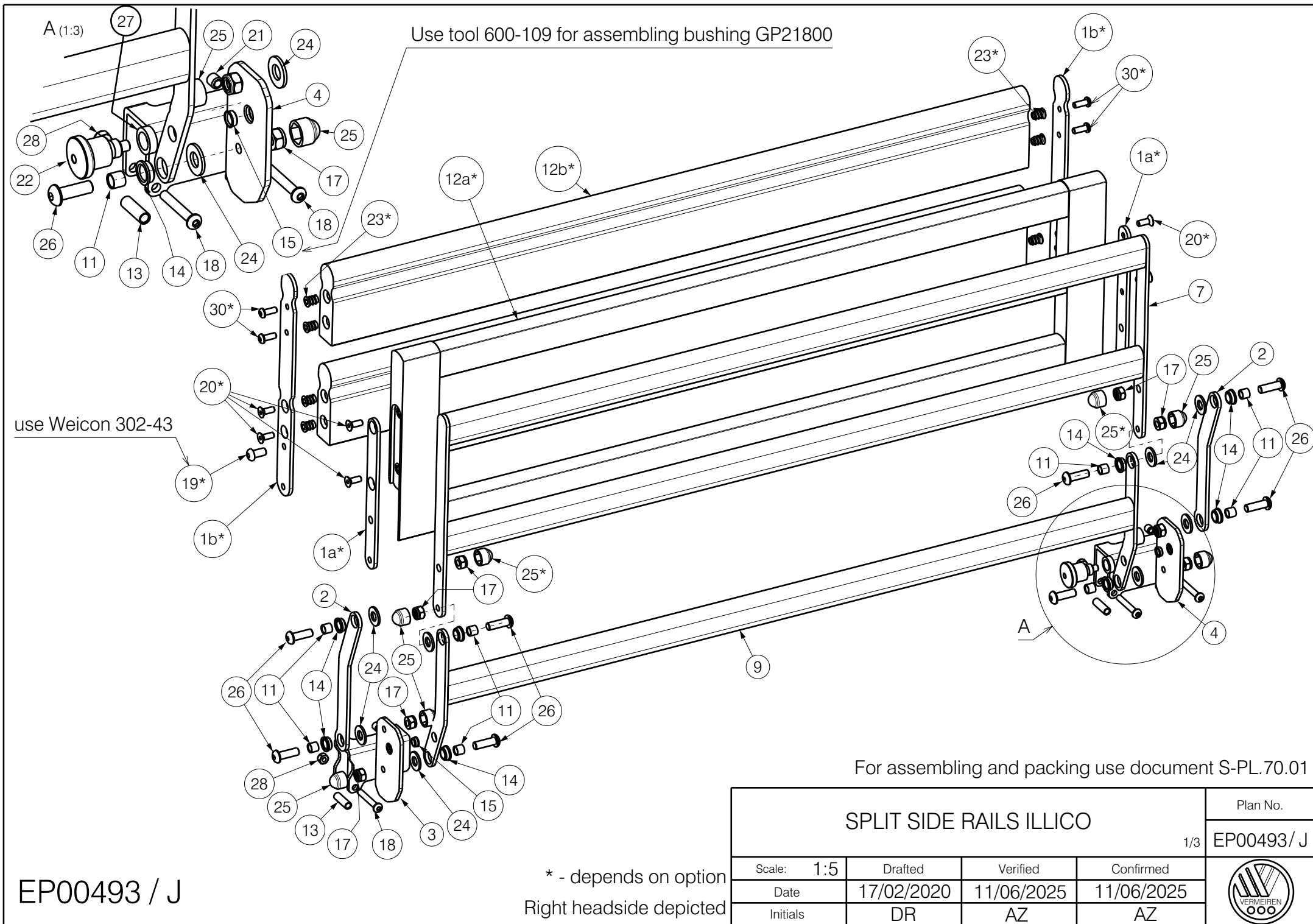

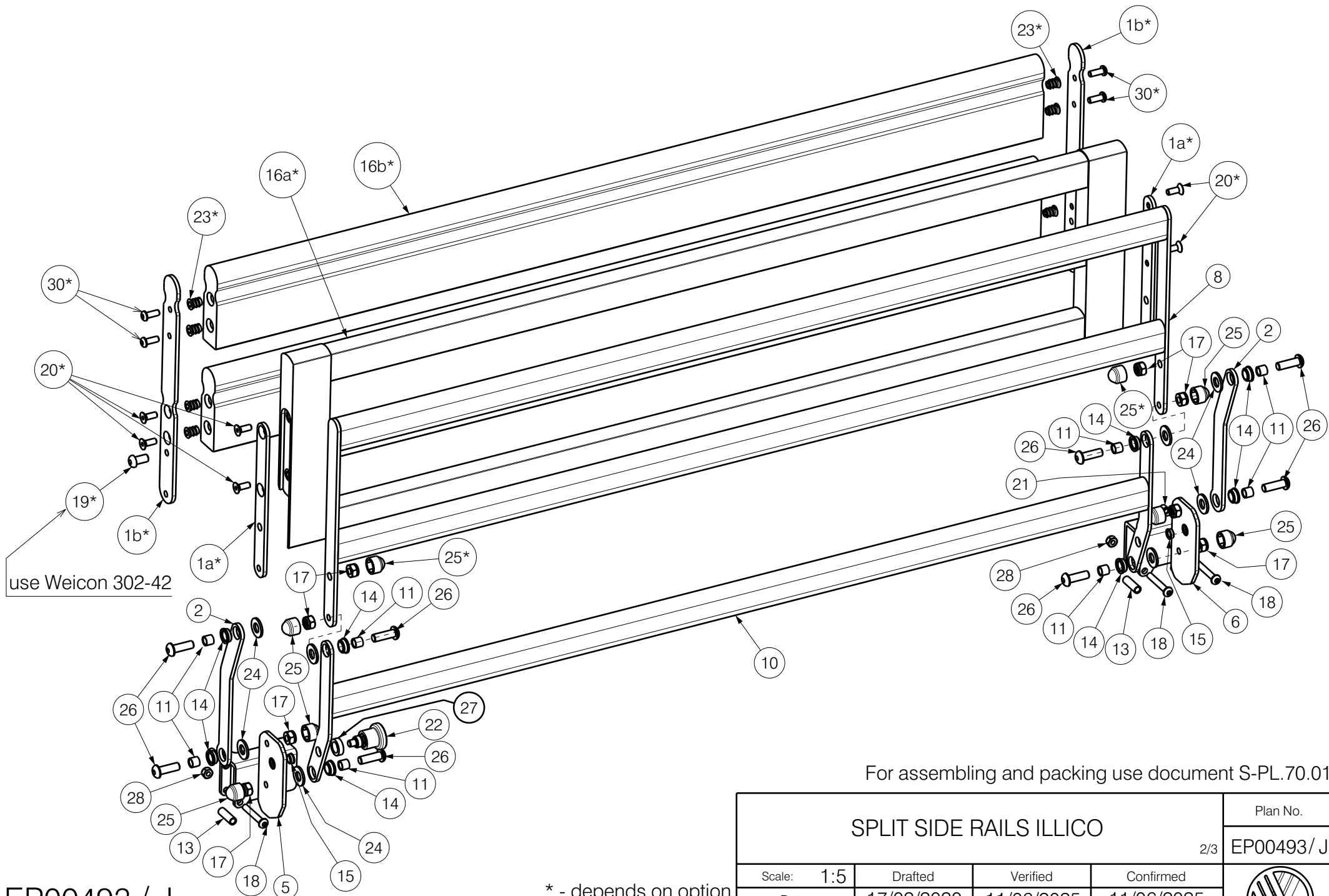

For assembling and packing use document S-PL.70.01

EP00493 / J

* - depends on option
Right footside depicted

SPLIT SIDE RAILS ILLICO				Plan No.
2/3				EP00493/ J
Scale: 1:5	Drafted	Verified	Confirmed	
Date	17/02/2020	11/06/2025	11/06/2025	
Initials	DR	AZ	AZ	

Material			
No.	Plan	Article	Quantity
1a*	YP07984	26037...	8
1b*	YP08012	26068...	8
2	YP07990	26038...	8
3	YP08013	26039... (L)	1
	YP08014	26040... (R)	1
4	YP08015	26041... (L)	1
	YP08016	26042... (R)	1
5	YP08017	26043... (L)	1
	YP08018	26044... (R)	1
6	YP08019	26045... (L)	1
	YP08020	26046... (R)	1
7	YP08021	26047...	2
8	YP08022	26048...	2
9	YP08023	26049...	2
10	YP08024	26050...	2
11	XP03050	GP23100	32
12a*	XP02891	HP00634 (beech)	2
	XP02931	HP00643 (bl. oak)	2
	XP03039	HP00667 (nat. oak)	2
	XP03040	HP00668 (wen. oak)	2
	XP03041	HP00669 (wal. oak)	2
12b*	XP02902	HP00641 (oxide)	4
	XP03197	HP00695 (beech)	4
13	XP03116	GP24700	12
14	XP02893	2994748	32
15	XP02894	GP21800	8
16a*	XP02895	HP00637 (beech)	2
	XP02932	HP00644 (bl. oak)	2
	XP03042	HP00670 (nat. oak)	2
	XP03043	HP00671 (wen. oak)	2
	XP03044	HP00672 (wal. oak)	2
16b*	XP02903	HP00642 (oxide)	4
	XP03198	HP00696 (beech)	4

Material			
No.	Plan	Article	Quantity
17*	Nut M8 Selflocking	1001682	32
18	M6x40	1006111	12
19*	M8x16	1991858	8
20*	M6x16 DIN7991	1994752	16
21	M8x8 DIN916	2996665	8
22	Blocker pin M12	2994540	4
23*	Insert M6x13	2995002	32
24	Washer M10 PVC	1001557	32
25*	Nut cap M8 grey	2994898	32
26	M8x25	1990501	32
27	XP03232	GP26300	4
28	Nut M6 Selflocking	1001680	12
29**	XP03129 box	U000430	1
30*	M6x16	1008459	16

* - depends on option

** - not depicted

For assembling and packing use document S-PL.70.01

EP00493 / J

SPLIT SIDE RAILS ILLICO				Plan No.
3/3				EP00493/ J
Scale: 1:5	Drafted	Verified	Confirmed	
Date	17/02/2020	11/06/2025	11/06/2025	
Initials	DR	AZ	AZ	