



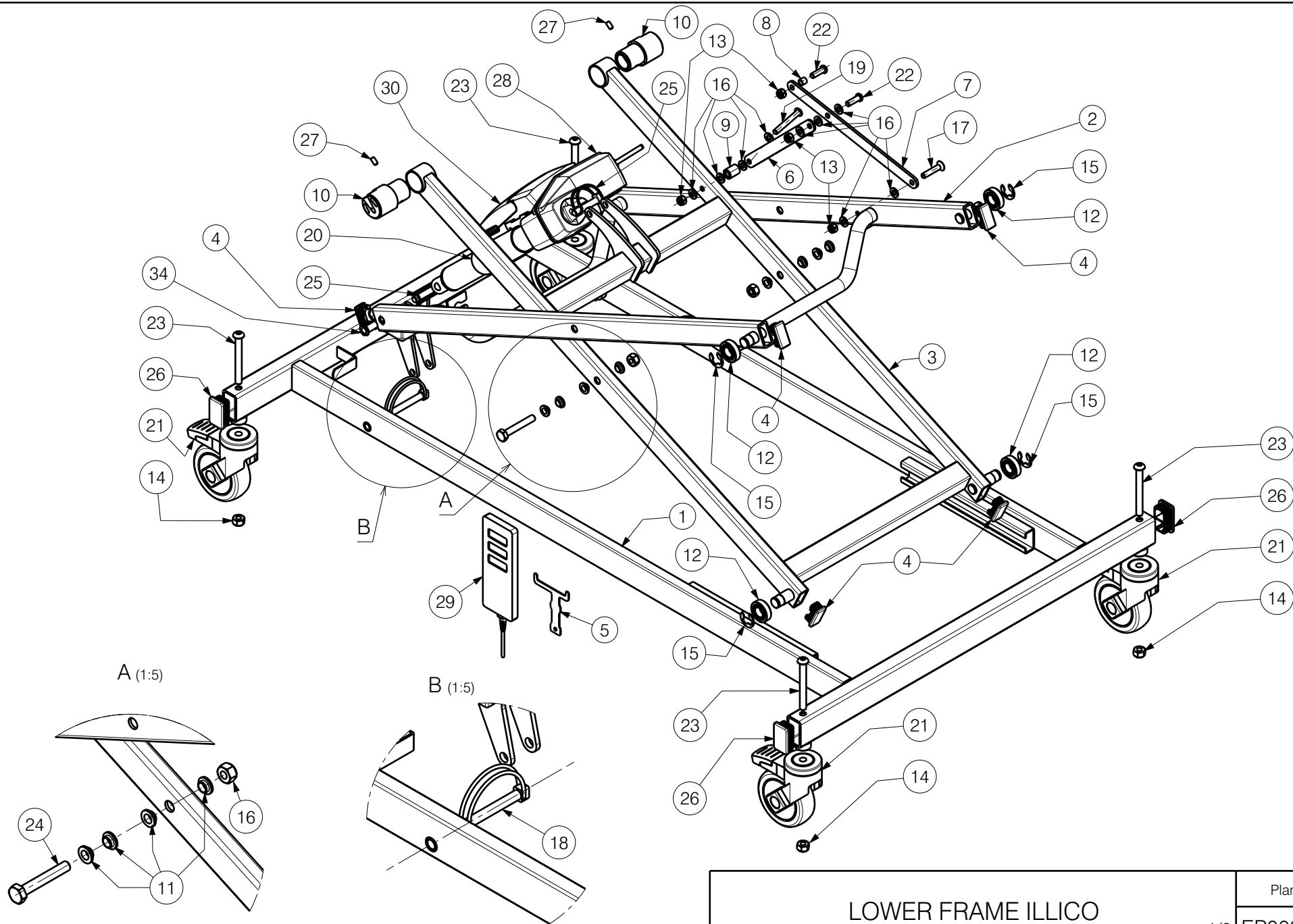

EP00247 / I

LOWER FRAME ILLICO				Plan No.
1/2				EP00247/I
Scale: 1:8	Drafted	Verified	Confirmed	
Date	19/10/2017	09/05/2025	09/05/2025	
Initials	DB	KK	AZ	

Material			
No.	Plan	Article	Quantity
1	YP05847	27838...	1
2	YP05862	27841...	1
3	YP05872	27843...	1
4	YP06097	2993740	6
5	YP06245	2795903	1
6	YP06445	27313...	1
7	YP06446	27314...	1
8	X001029	1375500	1
9	XP00871	2830300	1
10	XP01839	2993557	2
11	XP01931	2993584	8
12	Bearing D35xD17x10	1001409	4
13	Nut M8 DIN985	1001682	4
14	Nut M10 DIN985	1001683	6
15	Axle Ring DIN6799 D15	1003049	4
16	Washer M8 DIN125	1003199	9
17	M8x40 DIN7991	1008738	1
18	P00095127	2998591	2
19	M8x60	1900809	1
20	Control Box Support	1913262	1
21	Wheel D100	1914317	4
22	M8x25	1990501	2
23	M10x90	1998812	4
24	M10x65	1998862	2
25	Pen D10x50	1998885	2
26	Cap 30x50x3	2993583	4
27	Pin D6x12 DIN1481	2993738	2
28	Motor LA40 8000N	1914183	1
29	Remote Control	1904296	1
30	Control Box CA40	1912898	1
31*	EPR Cable	1902971	1
32*	Motor Cable LA40	1912903	1
33*	Cable number 1	1995835	1
34	P00100706	2998647	2
35*	Illico logo 25x99 CMYK	T001203	1

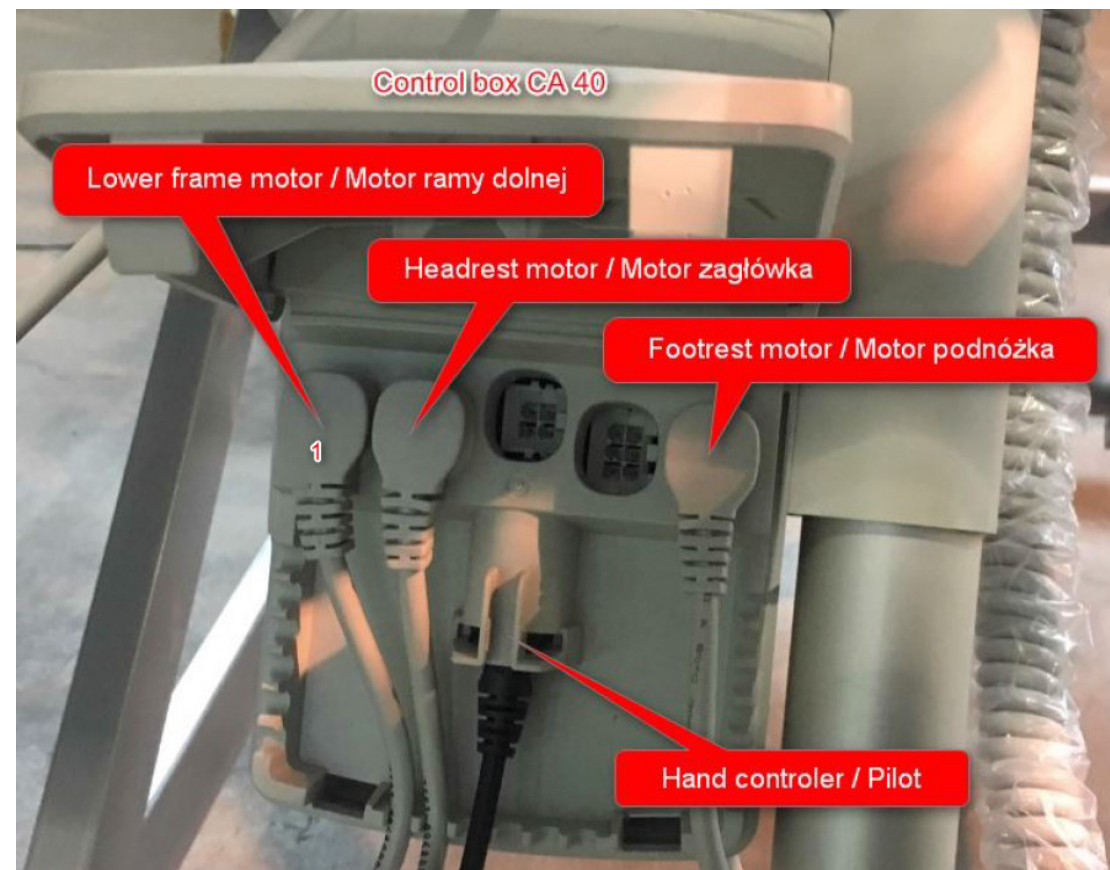

T001203 - Illico logo 25x95 CMYK
Dimensions : 25 mm height x 95 mm length
- Qty : 1 sticker
- Location : lower frame - foot panel side

LOWER FRAME ILLICO				Plan No.
2/2				EP00247/I
Scale: 1:8	Drafted	Verified	Confirmed	
Date	19/10/2017	09/05/2025	09/05/2025	
Initials	DB	KK	AZ	

EP00247 / I

* - not depicted / nie ujęto