


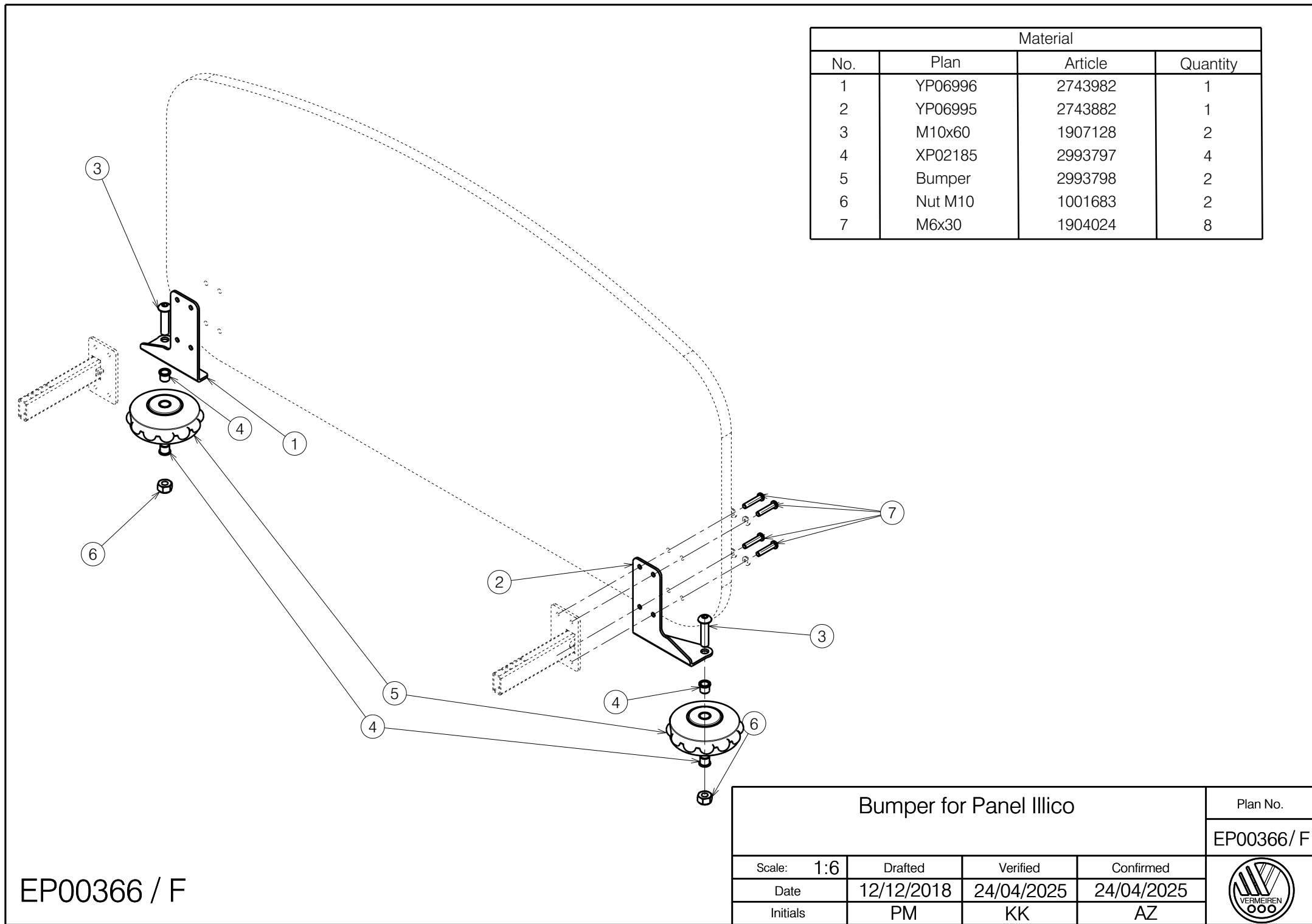

Material			
No.	Plan	Article	Quantity
1	YP06996	2743982	1
2	YP06995	2743882	1
3	M10x60	1907128	2
4	XP02185	2993797	4
5	Bumper	2993798	2
6	Nut M10	1001683	2
7	M6x30	1904024	8

EP00366 / F

Bumper for Panel Illico				Plan No.
				EP00366/ F
Scale: 1:6	Drafted	Verified	Confirmed	
Date	12/12/2018	24/04/2025	24/04/2025	
Initials	PM	KK	AZ	