


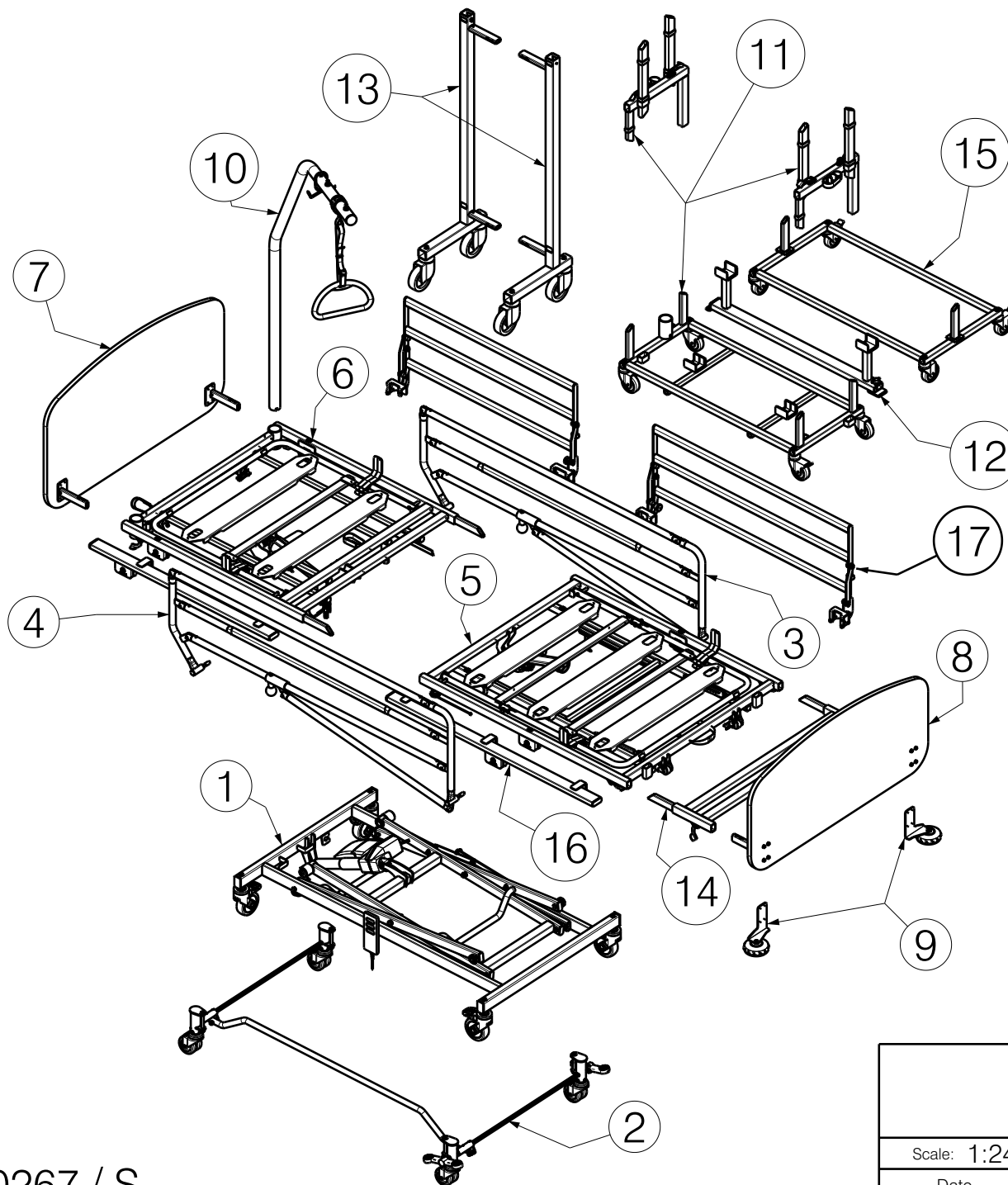

* - option
** - not depicted

EP00267 / S

EV ILLICO				Plan No.
				EP00267 / S
Scale: 1:24	Drafted	Verified	Confirmed	
Date	02/11/2017	11/06/2025	11/06/2025	
Initials	DB	KK	AZ	

Material			
No.	Plan/Article	Description	Quantity
1	EP00247	LOWER FRAME	1
2*	EP00357	CENTRAL BRAKE	1
	EP00359	CENTRAL BRAKE FOR ONE SIDE	1
3	EP00256	LEFT SIDE RAILS	1
	EP00280	LEFT SIDE RAILS 969	1
4	EP00257	RIGHT SIDE RAILS	1
	EP00280	RIGHT SIDE RAILS 969	1
5	E00070499	FOOT SIDE WITH METAL LATHS	1
	E00101041	FOOT SIDE WITH WOODEN LATHS	1
6	E00073286	HEAD SIDE WITH METAL LATHS	1
	E00101030	HEAD SIDE WITH WODDEN LATHS	1
7	EP00262	HEAD PANEL	1
	EP00272	HEAD PANEL 965	1
	EP00274	UPHOLSTERY HEAD PANEL	1
	EP00358	HEAD PANEL FULL	1
	EP00519	HAED SIDE PANEL / ILLICO VINTERNO	1
8	EP00263	FOOT PANEL	1
	EP00272	FOOT PANEL 965	1
	EP00273	UPHOLSTERY FOOT PANEL	1
	EP00358	FOOT PANEL FULL	1
	EP00520	FOOT SIDE PANEL / ILLICO VINTERNO	1
9*	EP00366	BUMPER FOR STANDARD PANEL	1
10	EP00268	SELF RAISING AID	1
11	EP00269	TRANSPORT SYSTEM	1
12*	EP00386	HORIZONTAL KIT FOR TRANSPORT SYSTEM	1
13*	EP00371	HORIZONTAL TRANSPORT SYSTEM	1
14*	EP00279	BED ELONGATION	1
15*	EP00411	VERTICAL TRANSPORT SYSTEM	1
16*	EP00473	WIDTH EXTENSION KIT	1
17*	EP00493	SPLIT SIDE RAILS	1
18**	EP00278	STICKERS	1

Material			
No.	Plan/Article	Description	Q-ty
19**	U000369	CARTON FOR BED WITH STANDARD PANEL	1
	U000367	CARTON FOR BED WITH PANEL 965	1
	U000386	CARTON FOR BED WITH CENTR.BR. STAND. PANEL	1
	U000387	CARTON FOR BED WITH CENTR.BR. PANEL 965	1
	U000424	CARTON FOR BED WITH STANDARD PANEL VINTERNO	1
	U000425	CARTON FOR BED WITH PANEL 965 VINTERNO	1
	U000426	CARTON FOR BED WITH CENTR.BR. STAND. PANEL VINT.	1
	U000427	CARTON FOR BED WITH CENTR.BR. PANEL 965 VINTERNO	1
20**	2993398	CARTON CORNER	4
21**	U000430	CARTON BOX FOR SPLIT SIDERAILS	1
	---	CARTON BOX FOR WIDTH EXTENSION	1
22**/**	EP00467	BEDS MATTRESS	1

EP00267 / S

* - option

** - not depicted

EV ILLICO				Plan No.
				2/2 EP00267 / S
Scale: ---	Drafted	Verified	Confirmed	
Date	02/11/2017	11/06/2025	11/06/2025	
Initials	DB	KK	AZ	