


\*21a = L34 / \*21b = L14

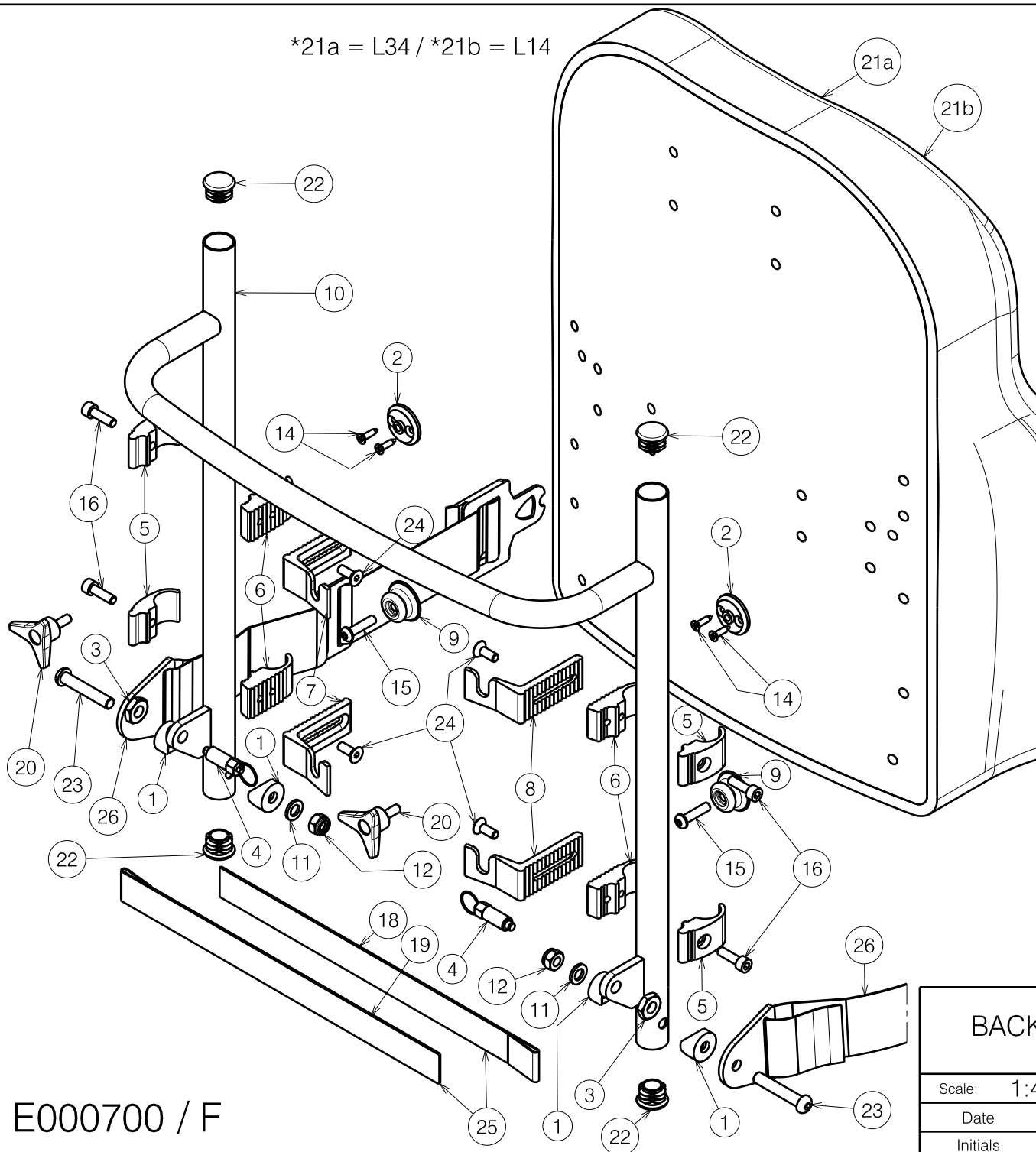

Material			
No.	Plan	Article	Quantity
1	X000862	1006463	4
2	X001052	G066600	2
3	X001398	1009689	2
4	---	2990301	2
5	X004820	1744532	4
6	X004821	1744632	4
7	X004822	1744732	2
8	X004823	1744832	2
9	X004861	1906454	2
10	Y018374	1768354 (40)	1
	Y018373	1768454 (45)	1
	Y018375	1768554 (50)	1
11	---	1001556	2
12	X001016	1001682	2
14	---	1005767	4
15	---	1006998	2
16	M6x20 DIN912	1001493	4
18	---	1009709	0.27
19	---	1009710	0.27
20	---	1009932	2
21a*	---	1906937 (40)	1
	---	1906939 (45)	1
	---	1906942 (50)	1
21b*	---	1906810 (40)	4
	---	1906812 (45)	2
	---	1906814 (50)	4
22	---	1990648	4
23	---	1994473	2
24	---	1994752	4
25	---	1009516	2X0.33
26	---	2993352	1

E000700 / F

BACK FRAME L14/L34 NAVIX MANUAL

Plan No.

E000700 / F

Scale: 1:4

Drafted

Verified

Confirmed

Date

28/06/2011

26/06/2019

26/06/2019

Initials

JG

TT

PJ

